

## BOSS™ STEAM CLEANER



## **BOSS™ STEAM CLEANER**

### PARTS:

1. STEAM CLEANER
2. HAND GUN
3. FOOT CONTROL
4. MEASURING CUP & A FUNNEL

### WARRANTY:

The BOSS™ STEAM CLEANER is for indoor use only. The steamer has **1 year warranty**, from the date of purchase against manufacturer defects. The steamer is not warranted against abuse, misuse or any other use than for which the steamer is intended. Removal of the steamer's outer covering voids the warranty.

### OPERATING MANUAL



**15031 PARKWAY LOOP #D, TUSTIN, CA. 92780**  
**TEL: (877) 258-7545 FAX : (714) 258-7584**  
**WWW.WHOLESALEDENTALSUPPLY.COM**

**15031 PARKWAY LOOP #D, TUSTIN, CA. 92780**  
**TEL: (877) 258-7545 FAX : (714) 258-7584**  
**WWW.WHOLESALEDENTALSUPPLY.COM**

## BOSS™ STEAM CLEANER

### WARNING:

Voltage: AC120V

Frequency: 60HZ

Power: 1450W

Water Capacity: Maximum 1600ml, Normal 1200ml.

1. Ensure that the electricity supply is the same as shown on the machine.
2. This steamer is not a **TOY** and should never be operated by children.
3. Steamer automatically heats and pressurizes when plugged in. It releases hot, pressurized steam when operated and caution should be used.
4. Steamer should never be operated when fill cap is not installed and hand tightened.
5. Fill with distill water only. Never add any chemical liquids such as detergents or alcohol or cleaners. Steamer is designed to safely operate with a maximum of 1300ml of water. **Never overfill the steamer as will cause sputtering and not function properly.**
6. Do not unscrew the safety lid when in use.
7. When in use outside of the machine will feel hot.
8. Steamer should never be refilled while still hot and under pressure. Never remove fill cap while unit is still hot and under pressure.
9. Steamer is made with a grounded electrical plus. Never bypass the ground wire.

The pressure gauge indicates the pressure of the tank inside. Normally the pressure gauge stays between 0—60 PSI. **When the pressure gauge exceeds over 60 PSI, this indicates water is low. (when in use, the pressure will go down quickly). Please stop using if the pressure gauge exceeds 80 PSI, shut off the power and release the pressure to add more water. When pressure is more than 80 PSI, Steam cleaner automatically will release some pressure through the cap, that indicates it is out off water.**

**15031 PARKWAY LOOP #D, TUSTIN, CA. 92780**

**TEL: (877) 258-7545 FAX : (714) 258-7584**

**WWW.WHOLESALEDENTALSUPPLY.COM**

## BOSS™ STEAM CLEANER

### OPERATING INSTRUCTION:

1. Place your steamer unit upright on a dry, flat, level surface. The edge of the sink or counter tops works well too.
2. Unscrew the cap, fill 1300 ml of distill water using the measuring cup provided. Do not over fill.
3. Replace fill-cap by screwing in and hand-tightening. Do not over-tighten the fill-cap.
4. Plug-in your steamer into grounded electrical outlet. Unit automatically heats and pressurizes.
5. Wait until **green** LED light turns off. This indicates unit is fully heated and pressurizes.
6. When it is necessary to refill the machine, first turn off the power switch and then release the remaining steam by pressing the button or use the foot switch until all the steam comes out. When pressure gauge is at zero then gently unscrew the fill cap.
7. When pressure goes down too fast that means it will run out of water. When there is no water inside the tank, the pressure gauge will be at 0 PSI and heating light will be off.

### Maintenance:

The steamer is designed to be very low maintenance. With moderate usage, your steamer should be cleaned annually. **Again distill water is recommended for steamer.** As tap water leaves a residue (hard water calcium build-up) that can be flushed out. Use any common bathroom cleaner intended to remove hard water stains. **(Make sure it can be used on metal).** Into a cool steamer unit, pour 1200ml of cleaner, let stand, swish around and pour out the cleaner. Rinse thoroughly with water before you use it. Never attempt to clean your steamer when it is in use or hot, wait till the unit has completely cooled down. Clean outer units with soft cloth and water, wipe it dry.

**15031 PARKWAY LOOP #D, TUSTIN, CA. 92780**

**TEL: (877) 258-7545 FAX : (714) 258-7584**

**WWW.WHOLESALEDENTALSUPPLY.COM**